

Newest Generation

Advanced Microneedling

SkinzRF

Driving topicals deeper into the dermis, resulting in enhanced. topical penetration up to 67% with the M and F Type needles.



Stepping Motor Type



Gold aPlating Needling




Mono Bipole




All body Treatment

Principles of treatment


SkinzRF

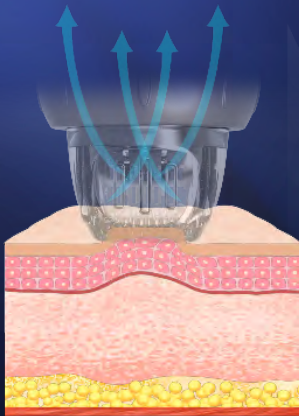


Auto Capture
Skin

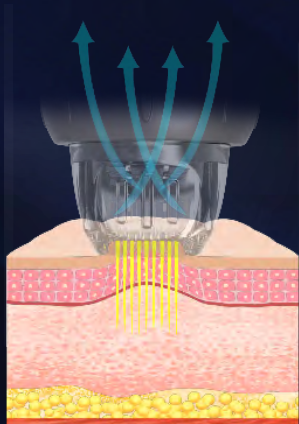

Creat Channels

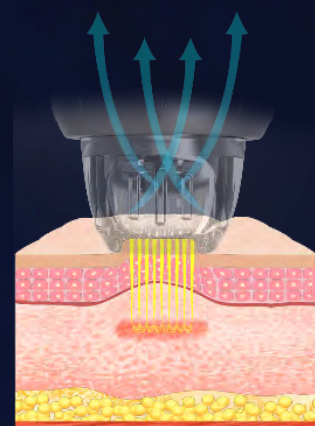

Release
Vacuum & Needle

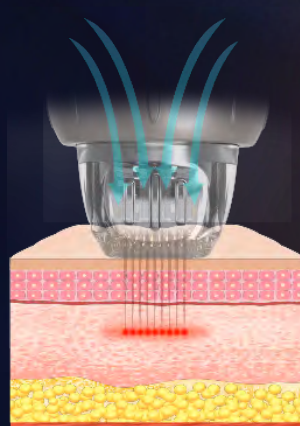


Finished



Treatment Range


SkinzRF

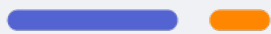

FACE

Fine Lines, Deep Wrinkles, Appearance of large pores, Acne Scars.

Fine Lines

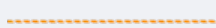

Deep Wrinkles



Large Pores



Acne Scars



BODY

Stretch Marks, Injury or surgery scar, Loose & Sagging Skin.



Stretch Marks




Injury Or Surgery Scar



Loose & Sagging Skin

Micro Needle Rf Applicator

— — — — —
SkinzRF

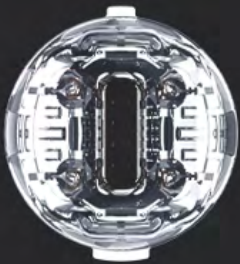

M-18

Stepping
Motor Type



M-25

Needle
Depth Control



M-49

Gold
Plating




M-81

Safety
Needle System

Fractional Rf Applicator

— —
SkinzRF

FRACTIONAL RF applicator for non-invasive wrinkle reduction suit for the patients fear of needle, no down time, recovery soon.

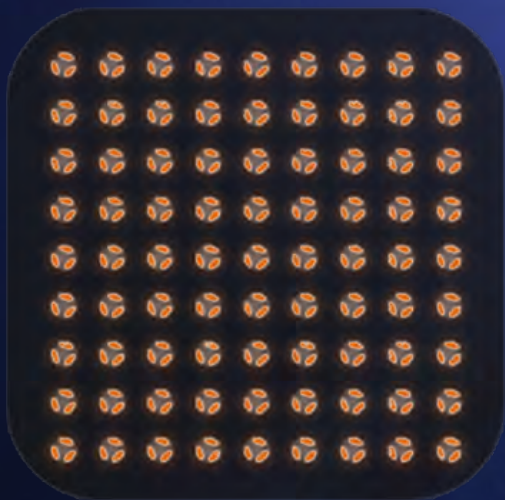

F-3*3

Eyes
Nose Area



F-5*5

Face Area




F-9*9

Body Area

Mono/Bipoles

SkinzRF

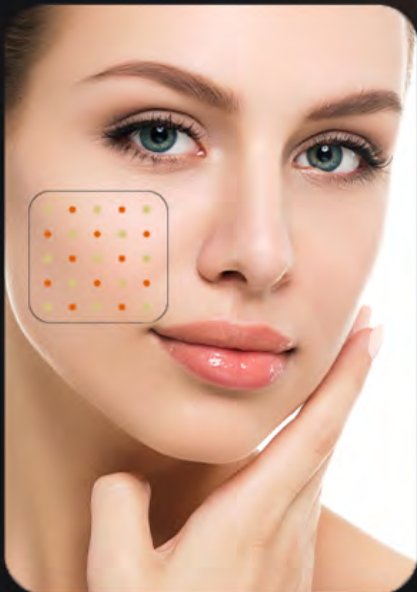

Face: Evenly Under Bipolar



50% percent needle is positive and the other 50% is negative pole. this make the energy only works on depth of the needle. This mode is more suitable for face treatment.



Body: Deeper Under Mono



All the needles on the tip is positive pole. The customer need to hold the negative pole in hand during the treatment. The RF energy can go to more deeper to skin than Bipole mode. So it's more suitable for body treatment.

Say Goodbye To Treating Pain

SkinzRF



Cooling Handle


Cooling Handle, used at before or after the treatment.

To have a better result and make customers more comfortable and reduce pain.



Parameter


SkinzRF

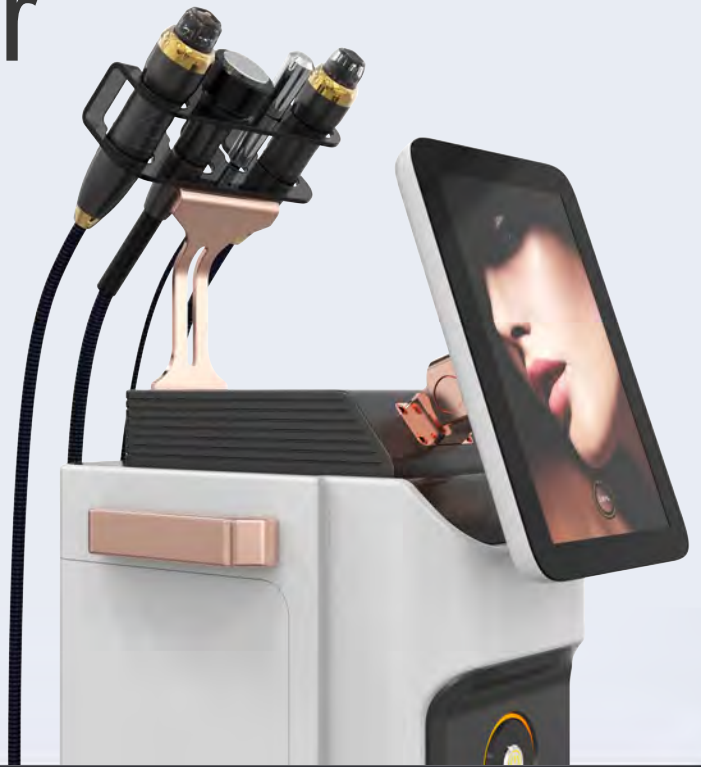

Products	SkinzRF machine
Power	max 50 watts
Handle	Fractional RF handle Micro Needle RF handle
Frequency	2MHz
Energy	1-10 Level
Pulse Duration	10-900ms
Needle Depths	0.4-4mm adjustable
Pressure Level	1-4
Working mode	SINGLE/AUTO
Mode	MONO & BIPOLAR
Cooling	1-3 Level
Tips	Insulated & Non-insulated
Invasive Tips	M-18, M-25, M-49, M-81
Non invasive Tips	F-3*3 , F-5*5, F-9*9

Treatment Effect

SkinzRF

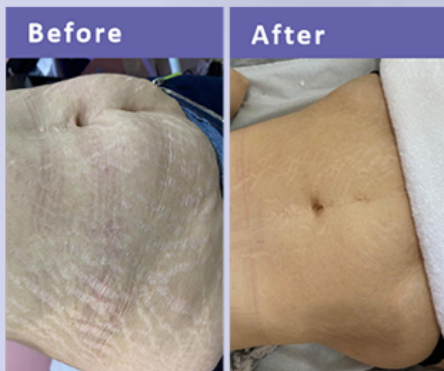

Stretch Marks - Age

Level 5 / 2.0 mm tip / 150 ms / 2 pass
3 weeks delay / 1 treatment times.



Scars From Injury

Level 4 / 1.0 mm tip / 140 ms / 2 pass
3 weeks delay / 3 treatment times.



Acne Pores

Level 4 / 1.5 mm tip / 120 ms / 1 pass
3 weeks delay / 1 treatment times.

Newest Generation microneedling RF

SkinzRF

